## Package Insert for BIOVER disposable microvascular clamps

### What are BIOVER disposable microvascular clamps for?

BIOVER disposable microvascular clamps are instruments which are used for all microsurgical procedures. They are used to occlude vessels during anastomosis, which is necessary as a result of vessel damage or thrombosis.

### What should the user know about BIOVER disposable microvascular clamps?

BIOVER disposable microvascular clamps can only be used **ONE TIME**.

Two types of microvascular clamps are available:

- **clamps for arteries are yellow** (single (1) and double (2))

- less – 1 mm dia.	single TKS-1	double TKS-2
- 1 mm – 2 mm dia.	single TKM-1	double TKM-2
- 2 mm – 4 mm dia.	single TKL-1	double TKL-2

- **clamps for veins are green** (single (1) and double (2))

- 0.2 to 1 mm dia.	single TKFV-1	double TKFV-2
- less – 1 mm dia.	single TKSV-1	double TKSV-2
- 1 mm – 2 mm dia.	single TKMV-1	double TKMV-2
- 2 mm dia.	single TKLV-1	double TKLV-2

- The BIOVER disposable microvascular clamps can be easily identified in the surgical site through their yellow (arteries) or green (veins) color. Yet the clamps do not reflect the light and are therefore easier on the eyes during long surgical procedures.
- BIOVER disposable microvascular clamps are radiolucent, except for the axis and the spring. Both types of disposable microvascular clamps can be seen very weakly in an MRI and in fluoroscopy.
- BIOVER disposable microvascular clamps deliver a precise and reliable closing pressure.  
The type of vessel (artery or vein) and the vessel diameter in mm indicates to the surgeon which type of disposable microvascular clamp should be used and which closing pressure is needed to be less traumatic to the vessel.

### How to use BIOVER disposable microvascular clamps?

Before using any type of disposable microvascular clamps, identify the type of vessel and vessel size to make sure that the right clamp with the correct clamping pressure is used.

- Choose between a single and a double BIOVER microvascular clamp.
- Use only products which are in the original and undamaged packaging to ensure that the sterility is guaranteed.
- Check the expiry date on the package to make sure that the product is still sterile.

The clamps are gamma sterilized and cannot be resterilized.

The manufacturer does not accept any liability for clamps which are resterilized by the user.

- Before closing the surgical site, check that all clamps have been removed.
- After any surgical procedure, the empty sterile pouches must be counted in order to determine the precise number of BIOVER disposable microvascular clamps used.

### WARNINGS

- **Never** use **damaged** clamps and/or **damaged** packaging.
- Use the BIOVER disposable microvascular clamps only **ONE TIME, SINGLE USE**.
- **Do not leave** any BIOVER disposable microvascular clamps in the wound after finishing the surgical procedures.
- **Do not use** the BIOVER vascular clamps for the central blood circulation and do not use for the central nervous system.

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BIOVER AG