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## IMPORTANT INFORMATION FOR THE USE IN HANDLING OF SURGICAL INSTRUMENTS CLIP APPLIERS

### CAUTION

Please read all information. Incorrect handling and care as well as misuse can lead to premature wear of surgical instruments. All persons using this device should be knowledgeable in the use and handling of surgical instruments.

### INITIAL USE OF NEW INSTRUMENTS

Every instrument must be cleaned and sterilized before it is used for the first time. Clip applicers were developed for sterilization by autoclave and are also compatible with STERRAD and STERIS method of sterilization.

### INSPECTION AND FUNCTIONAL CHECK

It is very important to carefully examine each surgical instrument for breaks, cracks or malfunctions before use. It is especially essential to check areas such as jaws as well as all movable parts. DO NOT USE DAMAGED INSTRUMENTS. Never attempt to make repairs yourself. Service and repairs should be referred to trained qualified persons only. Refer questions about repair to the manufacturer or your biomedical engineering department.

### DÉCONTAMINATION, CLEANING AND MAINTENANCE

Every surgical instrument must be disinfected and thoroughly cleaned after each use. Clean, inspect and test each instrument carefully. Sterilize all instruments before surgery. A proper cleaning and maintenance procedure will extend the life-time of the applicers. Special attention should be paid to the jaws. During cleaning and sterilization, standard clip applicer, for open surgery, must be opened (1) and (2). Laparoscopic clip applicers must be cleaned dismantled (refer to specific instruction for use supplied with the product).

- 1) Disengage the middle spring.
- 2) Open the handle completely.

### CAUTIONS

- Do not use corrosive cleaning agents. Cleaning solutions and rinses at or near a neutral pH(7.0) are best.
- Do not use abrasive cleaners.
- Only a soft brush should be used.
- Ultrasonic cleaners can be used.
- Rinse thoroughly with distilled water.
- Prepare for sterilization.

After cleaning and rinsing, dry instruments completely and carefully with compressed air.

### STANDARD STERILIZATION METHODS

Steam autoclave sterilization is recommended. Do not sterilize in hot air. Standard autoclave cycle: 134°C for 18 minutes. Other time and steam temperatures cycles may also be used. However, user must validate any deviation from the recommended time and temperature. Autoclave temperatures should not exceed 137°C (280°F).

### HANDLING

All surgical instruments should be handled with the greatest care, cleaned treated, sterilized and stored. This is especially important for jaws, which are very sensitive areas.

### WARRANTY

All Vitalitec International USA products are guaranteed to be free from defects in material and workmanship at the time of shipping. We cannot accept any liability for failure of products which have been modified in any way from their originals, for any failure due to misuse or application which is not in accordance with the designer intentions.

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Synovis Micro Companies Alliance, Inc.
(A subsidiary of Baxter International Inc.),